# MEDISANA®

home of wellness

DE **Massagesitz** MCH

GB **Massage seat** MCH

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PT **Asiento de massagem** MCH

NL **Massagekussen** MCH

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SE **Massagesäte** MCH

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## GB

- ① Power supply
- ② Massage seat
- ③ Motor arrangement ( 5 massage motors for back, waist, shoulders and thighs )
- ④ Connection for power unit and adapter cable
- ⑤ Control unit
- ⑥ I/O button (switches massage cushion on and off)
- ⑦ HEAT button (red LED illuminates when heater is switched on)
- ⑧ INTENSITY button (adjusts the massage intensity)
- ⑨ MODE button (selects massage type)
- ⑩ UPPER BACK button (Massage in the shoulder area)
- ⑪ LOWER BACK button (Massage in the back area)
- ⑫ HIPS button (Massage in the hip area)
- ⑬ THIGHS button (Massage on the thigh)
- ⑭ TIMER button (Massage time - preselection)
- ⑮ 15 status LED's
- ⑯ Remote control pocket
- ⑰ Car adaptor cable

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## IMPORTANT INFORMATION! RETAIN FOR FUTURE USE!

Read the instruction manual carefully before using this device, especially the safety instructions, and keep the instruction manual for future use. Should you give this device to another person, it is vital that you also pass on these instructions for use.

### *Explanation of symbols*

This instruction manual belongs to this device. It contains important information about starting up and operation. Read the instruction manual thoroughly. Non-observance of these instructions can result in serious injury or damage to the device.

#### **WARNING**

These warning notes must be observed to prevent any injury to the user.

#### **CAUTION**

These notes must be observed to prevent any damage to the device.

#### **NOTE**

These notes give you useful additional information on the installation or operation.

Protection category II

**LOT**

Lot number

Manufacturer

### Safety Instructions

#### **about the power supply**

- Before connecting the device to your power supply, please ensure that the supply voltage stated on the rating plate is compatible with your mains supply.
- Only connect the mains plug to the wall socket when the device is switched off.
- Only use the mains adapter that is provided with the massage seat.
- Keep the power supply, the cable, the control unit and the massage seat away from heat, hot surfaces, moisture and liquids. Never get hold of the power supply with wet hands.
- Do not take hold of a device if it falls into water. Pull out the power plug and the power supply unit from the socket immediately.
- Always unplug the equipment immediately after use.
- To disconnect the unit from the power supply, always pull the plug out of the socket. Do not pull on the cable!
- Do not carry, pull or twist the massage seat on the cable, the power supply unit or control unit.
- You must not continue to use the unit if the lead or the mains adaptor are damaged. For safety reasons, these parts may only be replaced by authorized service personnel. In order to prevent hazards, always send the unit in to the service centre.
- Ensure that the cable will not cause tripping. The cable must not be kinked, trapped or twisted.

#### **specific groups of people**

- This appliance can be used by children aged from 8 years and above and persons with reduced physical, sensory or mental capabilities or lack of experience and knowledge if they have been given supervision or instruction concerning use of the appliance in a safe way and understand the hazards involved.
- The surface of the appliance is hot. People who are insensitive to heat must be careful when using the appliance.
- Children must be supervised to ensure that they not play with the device.
- Do not use this unit to supplement or replace medical treatment. Chronic complaints and symptoms may be made worse.
- You should refrain from using the massage seat cover **MCH**, or should consult your doctor before using it, if
  - you are pregnant,
  - you have a cardiac pacemaker, artificial joints or electronic implants,
  - you suffer from one or more of the following illnesses and complaints: Circulation disorders, varicose veins, open wounds, bruises, broken skin, inflammation of the veins.
- If you experience any pain or discomfort during the massage, stop immediately and consult your doctor.
- Never leave the unit unattended if it is connected to the power supply.
- Take care when using the heater. Do not use if you have circulatory problems or on areas not sensitive to heat. **Risk of burning!**
- Consult your doctor if you have any questions concerning the therapeutic use of massage seats.
- If you have unexplained pain, are receiving treatment from a doctor and/or use medical equipment, consult your doctor before using the massage seat.

**before using the unit**

- Always carefully inspect the power supply, the cable, the control unit and the massage seat for damage before use. Do not attempt to use a defective unit.
- Do not use the unit if there are visible signs of damage on the unit or the cable components, the unit does not work properly or the seat or the control unit have been dropped or become damp. In order to prevent hazards, always send the unit in to the service centre.

**operating the unit**

- The massage seat must only be used for its intended purpose as described in the instruction manual.
- Using the mat for any other purpose invalidates the warranty.
- Never leave the unit unattended if it is connected to the power supply.
- The maximum operating time for one session is 60 minutes.
- Do not allow the massage cushion to come into contact with pointed or sharp objects.
- Never put the unit down next to an electric stove or other sources of heat.
- Never use the massage seat if it is folded up or crumpled.
- Do not stand on the unit.
- Do not touch the power supply unit when you are standing in water and always take hold of the plug with dry hands.

**The device is not to be used for commercial use or medical purposes.**

**If you have health concerns, consult your doctor before using the massage cushion.**

**SAFETY IN ROAD TRAFFIC**

**Safety has the utmost priority in road traffic. The driver may not use the massage cushion when driving.**

**Only use the massage cushion as a passenger or in a parked vehicle.**

**Only use the massage cushion indoors or in the vehicle.**

**Do not use the massage seat in damp rooms (e.g. when bathing or showering).**

**maintenance and cleaning**

- The unit is maintenance free. If a fault does occur, just check whether the mains power adapter is properly connected.
- You may only clean the device yourself. If the device malfunctions please do not repair it yourself as this leads to the automatic expiry of all guarantee rights. Ask your specialist dealer and have repairs carried out by the service.

- Do not use the unit if there are visible signs of damage on the unit or the cable components, the unit does not work properly or the massage seat or the control unit have been dropped or become damp. In order to prevent hazards, always send the unit in to the service centre.
- Do not immerse the device in water or other fluids.
- If, nonetheless, liquid should penetrate the device, unplug the mains cable immediately.
- Cleaning and user maintenance shall not be made by children without supervision.

**Do not wash!**

**Do not dry clean!**

## 2 Useful Information

### Thank you very much

Thank you for your confidence in us and congratulations!  
You have acquired a **MEDISANA** quality product with this massage seat cover **MCH**. This unit is designed for massaging the back and upper thigh areas. We recommend that you carefully read the following information on use and care of the product so that you experience its benefits and enjoy using your **MEDISANA** massage seat cover **MCH** for many years.

### 2.1 Items supplied and packaging

Please check first of all that the device is complete and is not damaged in any way. In case of doubt, do not use it and contact your dealer or your service centre.

The following parts are included:

- 1 **MEDISANA** Massage seat **MCH**
- 1 Control unit
- 1 Power supply
- 1 Car adaptor cable
- 1 Instruction manual

The packaging can be reused or recycled. Please dispose properly of any packaging material no longer required.  
If you notice any transport damage during unpacking, please contact your dealer without delay.

### **WARNING**

**Please ensure that the polythene packing is kept away from the reach of children! Risk of suffocation!**

### 3.1 Preparation

- Place the massage seat **2** on a high chair or armchair (Car seat) and attach it to this by means of the fasteners on the back.
- Connect the power supply unit cable **1** or the adaptor cable **17** to the connection on the massage mat.
- Plug the power supply unit **1** into the electric socket or plug on the adaptor cable **17** into the cigarette lighter.
- Now sit on the seat **2** and familiarise yourself with the controls.

### 3.2 Operating the unit

The massage seat has four massage zones for specific treatment. These are in the main areas of tension (shoulders, back, waist and thighs). Different massage programmes with different speeds can be selected for each massage zone.

The timer switches the device off automatically once the selected time of 15, 30 or 60 minutes has elapsed.

### 3.3 Buttons on the control unit

- I/O **15** (Voltage supply):  
**Switch on:** Press the I/O button.  
**Switching off:** Press the I/O button again. To reset after timer shuts off, push either I/O or HEAT button **7**.

#### CAUTION

**Make sure that the maximum operating time of 60 minutes is not exceeded.**

- HEAT **7** :  
Press the HEAT button to switch the heater on. When the heater is on the red LED illuminates. The waist area is heated. Pressing the HEAT button again switches the heater off. The heat function can operate separately or combined with MASSAGE function.
- INTENSITY **8** (Intensity):  
Press the INTENSITY button to set the required intensity.
- MODE **9** (Massage type):  
Select one of the following functions, 1 to 5.  
**AUTO:** This is the standard massage type when the unit is switched on. In Auto mode, the device runs through all the types of massage one after the other automatically. Each press of the MODE button switches to the next massage type.  
**KNEADING:** Press the MODE button once after switching on. This massage type is like a Shiatsu massage.  
**PULSATE:** Press the MODE button twice after switching on. The massage motors pulsate alternately.  
**TAPPING:** Press the MODE button three times after switching on.

**ROLLING:** Press the MODE button four times after switching on. With this massage type the motors are switched in such a way that a wave-like massage feeling is generated.

**AUTO:** \_ \_ \_

- **MASSAGE ZONES:**  
You can choose between 4 massage zones in any combination.

- **10** Shoulder area (UPPER BACK)
- **11** Back area (LOWER BACK)
- **12** Hip area (HIPS)
- **13** Thighs (THIGHS)

By pressing the appropriate button, a massage zone can be targeted or switched off. If a massage zone is activated, the LED for it lights up.

- **TIMER 14:**  
Press the TIMER button to set one of the preset massage times of 15, 30 or 60 minutes. The LED lights up.

#### 4.1 Cleaning and maintenance

- Before cleaning the device, make sure that it is turned off and the mains plug is removed from the socket. Allow the unit to cool down.
- The massage seat mat must only be cleaned with a damp sponge. Never use brushes, aggressive cleaning agents, petrol, thinners or alcohol. Rub the seat dry with a clean dry cloth.
- When cleaning, never immerse the unit in water and make sure that no water gets into the unit.
- Wait until the device is completely dry before using it again.
- Undo any kinks in the cable.
- You should ideally store the device in its original packaging and keep it in a clean, dry place.

#### 4.2 Disposal

This product must not be disposed of together with domestic waste. All users are obliged to hand in all electrical or electronic devices, regardless of whether or not they contain toxic substances, at a municipal or commercial collection point so that they can be disposed of in an environmentally acceptable manner. Consult your municipal authority or your dealer for information about disposal.

#### 4.3 Technical Data

Name and model	: MEDISANA Massage seat <b>MCH</b>
Voltage supply	: 100-240 V~ 50/60 Hz 500 mA
Power supply unit	: 12 V= 800 mA IVP1200-0800WG
Car adaptor	: 12 V= 3 A (E9) E9-10R-04.1145
Power consumption	: approx. 9.6 W
Operating time	: max. 60 minutes
Timer	: after approx. 15, 30 and 60 minutes
Operating conditions	: only use in dry rooms
Storage conditions	: in a cool, dry place
Dimensions	: approx. 110 x 48 cm
Weight	: approx. 1.1 kg (without power supply)
Power cable length	: approx. 1.8 m
Adapter cable length	: approx. 1.8 m
Article No.	: 88935
EAN Code	: 40 15588 88935 6

**In accordance with our policy of continual product improvement, we reserve the right to make technical and optical changes without notice.**

The current version of this instruction manual can be found under [www.medisana.com](http://www.medisana.com)

**Warranty and  
repair terms**

Please contact your dealer or the service centre in case of a claim under the warranty. If you have to return the unit, please enclose a copy of your receipt and state what the defect is.

The following warranty terms apply:

1. The warranty period for **MEDISANA** products is three years from date of purchase. In case of a warranty claim, the date of purchase has to be proven by means of the sales receipt or invoice.
2. Defects in material or workmanship will be removed free of charge within the warranty period.
3. Repairs under warranty do not extend the warranty period either for the unit or for the replacement parts.
4. The following is excluded under the warranty:
  - a. All damage which has arisen due to improper treatment, e.g. non-observance of the user instructions.
  - b. All damage which is due to repairs or tampering by the customer or unauthorised third parties.
  - c. Damage which has arisen during transport from the manufacturer to the consumer or during transport to the service centre.
  - d. Accessories which are subject to normal wear and tear.
5. Liability for direct or indirect consequential losses caused by the unit are excluded even if the damage to the unit is accepted as a warranty claim.

**MEDISANA GmbH**  
Jagenbergstraße 19  
41468 NEUSS  
GERMANY

E-Mail: [info@medisana.de](mailto:info@medisana.de)  
Internet: [www.medisana.de](http://www.medisana.de)

The service centre address is shown on the attached leaflet.

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