


Instructions for Use **Ambu[®] ResCue Mask[™]**

Ambu A/S
Baltorpbakken 13
DK-2750 Ballerup, Denmark
T +45 72 25 20 00
F +45 72 25 20 50
www.ambu.com

492.4201.40 - V05 - 2017/08 - Printed in China - Ambu A/S, TCC 10333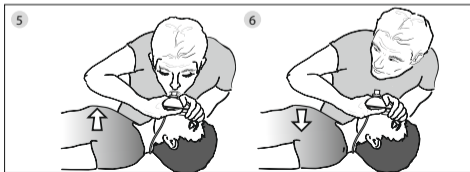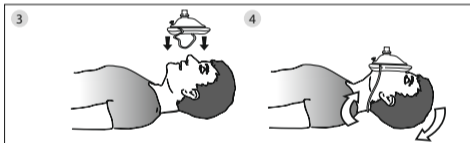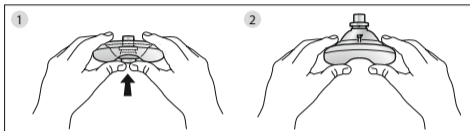

EN	This product is not made with natural rubber latex nor phthalates	LT	Gaminio sudėtyje nėra nei natūralaus latekso, nei ftalatų
BG	Този продукт не е произведен с естествен каучуков латекс или фталати	LV	Šis izstrādājums nav veidots no dabīgās gumijas lateksa un ftalātiem
CS	Tento výrobek není vyroben z přírodního latexu nebo ftalátů	NL	Dit product is niet vervaardigd met natuurlijke rubberlatex of ftalaten
DA	Denne produkt er hverken fremstillet med naturgummilatex eller ftalater	NO	Denne produktet er ikke fremstilt av naturlig gummilatex eller ftalater
DE	Dieses Produkt enthält weder natürliches Gummilatex noch Phthalate	PL	Ten produkt nie został wykonany z lateksu z kauczuku naturalnego ani z ftalanów
EL	Αυτό το προϊόν δεν έχει κατασκευαστεί με φυσικό ελαστικό λάτεξ ή φθαλκικές ενώσεις	PT	Este produto não é fabricado com borracha de látex natural nem ftalatos
ES	Este producto no está fabricado con látex ni ftalatos	RO	Acest produs nu este fabricat din latex din cauciuc natural sau din ftalați
ET	Toode on valmistatud ilma loodusliku kummilateksi ja ftalaatideta	RU	В производстве данного продукта не используются фталаты и латекс из природного каучука
FI	Tämä tuote ei sisällä luonnonkumilateksia eikä ftalaatteja	SK	Tento výrobok nie je vyrobený z prírodného gumeného latexu ani ftalátov
FR	Ce produit n'a été fabriqué ni avec du latex de caoutchouc naturel ni avec des phthalates	SL	Ta izdelek ni narejen iz lateksa naravnega kavčuka ali ftalátov
HR	Ovaj proizvod nije napravljen s lateksom od prirodne gume niti ftalatom	SV	Produkten innehåller inte naturgummilatex eller ftalater
HU	Ez a termék nem természetes latexgumból, nem ftalátokból készült	TR	Bu ürün doğa! kauçuk lateks veya ftalat kullanılarak imal edilmemiştir
IT	Il prodotto non è realizzato con lattice di gomma naturale né con ftalati	ZH	本产品不含天然乳胶和邻苯二甲酸盐
JA	本製品に天然ゴムラテックスあるいはフタル酸エステルは使用されていない		

Contents

Page

English (Instructions for use)	3
България (Ръководство за употреба)	9
Česky (Návod k použití)	17
Dansk (Brugsanvisning)	23
Deutsch (Bedienungsanleitung)	29
Ελληνικά (Οδηγίες χρήσης)	37
Español (Instrucciones de uso)	45
Eesti (Kasutusjuhised)	51
Suomi (Käyttöohje)	57
Français (Mode d'emploi)	63
Hrvatski (Upute za upotrebu)	71
Magyar (Használati útmutató)	77
Italiano (Istruzioni per l'uso)	83
日本語 (使用法)	91
Lietuviškai (Naudojimo instrukcijos)	97
Latviski (Lietošanas instrukcija)	103
Nederlands (Gebruiksaanwijzing)	109
Norsk (Bruksanvisning)	115
Polski (Instrukcja użytkowania)	121
Português (Instruções de utilização)	129
România (Instrucțiuni de utilizare)	135
Русский (Инструкция по применению)	143
Slovenčina (Návod na použitie)	151
Slovenščina (Navodila za uporabo)	157
Svenska (Bruksanvisning)	163
Türkiye (Kullanım kılavuzu)	169
中文 (使用说明)	175

Contents	Page
1. Use on adults and children.....	4
2. Use on infants.....	5
3. Use with oxygen (optional).....	5
4. Use with other ventilation equipment.....	6
5. Reuse.....	6
6. Cleaning.....	6
7. Technical specifications.....	7
8. Material.....	7
9. Temperature limits.....	7

The Ambu® ResCue Mask™ enables performance of mouth to mask ventilation. Additionally it provides a physical barrier between the rescuer and the victim, eliminating direct contact of the rescuer's lips with the unknown subject.

Proper training in ventilation and CPR should be obtained before using this device. Practice using your ResCue Mask on training manikins so that you are ready for any emergency that might occur. In the event that you do not have your Ambu ResCue Mask start CPR without delay.

After removal of the valve, the Ambu ResCue Mask can be attached to a resuscitator if available.

Start CPR without delay

CAUTIONS

- Ambu ResCue Mask is intended to be only used by trained users.
- Make sure that the valve is properly attached to the mask before starting ventilation. If not properly attached it may result in insufficient ventilation. If the one-way valve is replaced after the mask has been used, the replacement valve should only be inserted halfway into the mask and then be pressed fully into the mask just before starting the ventilation, in order to secure proper attachment.

1. Use on adults and children

Carefully inspect all parts of damage or excessive wear and replace, if necessary.

- Remove the mask/valve assembly from case. Push out the soft dome with your fingers and ensure that the one-way valve is in place.
- Put on the protective gloves if available.
- Clear the airway of any liquid (blood, saliva and gastric content).
- Place the mask over the patient's mouth and nose. Position the elastic rubber band around the head if available.* Apply the mask to the patient's face using the thumbs of both hands. Lift the jaw into the mask with the remaining fingers by exerting pressure behind the angles of the jaw (jaw thrust). At the same time, press the mask onto the face with the thumbs to make a tight seal.

- Blow through the inspiratory valve and watch the chest rise. Stop inflation and observe the chest falling. The seal between the face and mask can be reduced by adjusting the contact pressure, altering the position of the fingers and thumbs, or increasing the jaw thrust.
- Continue rescue breathing until the patient responds or you are relieved by a higher medical authority.

* The Ambu ResCue mask comes both with and without an elastic rubber band.

NOTE

If the patient vomits; remove the ResCue Mask and clean both the patient's airway and the mask with your gloved fingers. If you should contaminate either the valve mouthpiece or any of your exposed skin surfaces, use the alcohol wipes to clean these areas immediately. Resume rescue breathing promptly.

2. Use on infants

The Ambu ResCue Mask may be reversed to better fit an infants face. Place the nose section of the mask under the infants chin. Seal the ResCue Mask as described under the Adult/Child section and ventilate using the previously described technique.

The one-way valve of Ambu ResCue Mask can be connected to a face mask with 15mm standard male connector.

3. Use with oxygen (optional*)

The Ambu ResCue Mask may be used with supplemental oxygen. Attach the oxygen tubing to the optional oxygen inlet and set the flow rate. A head strap is included on all models that have the optional oxygen valve. If the patient is spontaneously breathing, use the head strap to position the ResCue Mask on the patient's face. If the patient is NOT breathing, begin mouth to mask ventilation immediately. The use of supplemental oxygen to the Ambu ResCue Mask will increase the delivered oxygen to the patient. The rescuer will not be affected by the use of supplemental oxygen.

*ResCue Mask with Oxygen Inlet is Rx Only in the USA.

4. Use with other ventilation equipment

The Ambu ResCue Mask may be used with Bag-Valve Mask ventilation devices. Simply detach the valve/filter assembly and attach the ResCue Mask to the device.

5. Reuse

The mask consists of a reusable mask and a single use valve. The valve cannot be cleaned. So it needs to be discarded after used and must be replaced by a new valve before reusing the mask.

CAUTIONS

- Don't place the used mask in the hardcase or softcase before cleaning the mask.
- After cleaning the ResCue Mask should be stored inside the case to keep clean until next use.

6. Cleaning

The Ambu ResCue Mask is reusable and easy to clean. Hand wash cleaning procedure:

- Prior to cleaning remove the valve containing the filter by twisting and pulling. Discard the valve. Remove and discard the elastic rubber band. Open the white oxygen inlet port.
- Prepare the detergent bath dissolving conventional dish-washing detergent in hot water from the tap (40-60 °C) at 1 ml per gallon (1 ml per 3.38 liters).
- Fully immerse the mask in the detergent bath placing a weight on the device to keep it submerged in the solution for 10 minutes. Agitate the mask while it is fully immersed to allow complete penetration of detergent to all parts of the mask.
- Thoroughly clean the mask with a soft bristle brush until all visible dirt is removed, paying close attention to the inside of the mask and the internal union between the cuff and the mask.
- Using a sterile syringe flush the detergent solution through the oxygen inlet lumen. A pipe cleaner or similar may be used in hard to reach areas.
- Repeat steps 2 to 5.
- Rinse the mask for one minute with hot water from the tap (40-60 °C) ensuring water is flushed through the oxygen inlet port.
- Let the device air-dry at room temperature for 10 minutes.
- After cleaning the ResCue mask should be visually inspected. If any damages are seen the ResCue Mask should be discarded.

Clean the internal side of the case with alcohol 95% and let dry before placing the clean mask back in the case.

CAUTIONS

- Do not boil, steam or gas autoclave this device!

NOTE

It is the user's responsibility to evaluate the effectiveness of any deviation from the instructions provided.

7. Technical specifications

Ambu ResCue Mask is in conformity with Council Directive 93/42/EEC concerning Medical Devices.

Connector Dimensions:

- Mask: able to be connected to a standard 22 mm male connector
Valve: able to be connected to a standard 15 mm male connector

Dimensions: 138 mm x 113 mm x 50 mm

Weight: Approximately 117 g

8. Material

- Mask: Polyvinylchloride
Valve: Styrene-Butadiene Copolymers, Polypropylene and silicone
Hard case: Polypropylene
Soft case: Polyurethane, Ethylene Vinyl Acetate and Micro Fabric
Bag: Nylon

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