


Xprecia Stride™ Coagulation Analyzer

► For complete instructions, see the Xprecia Stride User Guide and the instructions for use that came with your test strips and controls.

Patient Test




1. Scan Patient ID (optional)
2. Scan strip
3. Insert strip
4. Scan strip vial (for new lot number)
5. During warm up, prepare patient for fingerstick
6. Hold level
7. Apply sample
8. Test in progress
9. View result
10. Eject strip
11. Clean Stride

Quality Control Test



IMPORTANT

◀ Prepare Xprecia™ System(s) PT Controls in advance and allow to stand for 5 minutes. Use the reconstituted control solution within 25 minutes, when stored at 15–25°C (59–77°F). For complete instructions, see the Xprecia Stride User Guide and the instructions for use that came with your controls.



1. Scan control solution bottle
2. Scan strip
3. Insert strip
4. Scan strip vial (for new lot number)
5. Gather controls
6. Hold Level
7. Apply control sample
8. Test in progress
9. View result
10. Eject strip
11. Clean Stride





i For complete cleaning instructions, see the Xprecia Stride User Guide




Reference Information

Settings


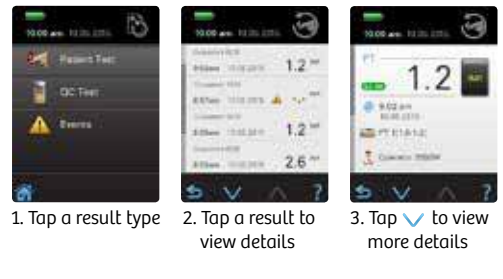



Settings – Icon Key

-  **Clock**
Change the time.
-  **Date**
Change the date.
-  **Analyzer**
Customize the volume, brightness, battery type, default settings, and how results are displayed.
-  **Administrator**
Customize the administrator settings for optional Operator ID (OID), Patient ID (PID), and Login.

-  **Language**
Change your Language settings.
-  **Information**
View the software version number.
-  **Tutorial**
Learn how to perform a patient test, QC test, or hold Stride.

Recall Results

1. Tap a result type
2. Tap a result to view details
3. Tap ✓ to view more details

Xprecia Stride and all associated marks are trademarks of Siemens Healthcare Diagnostics Inc., or its affiliates. All other trademarks and brands are the property of their respective owners.

Product availability may vary from country to country and is subject to varying regulatory requirements. Please contact your local representative for availability.

Siemens Healthineers Headquarters

Siemens Healthcare GmbH
Henkestr. 127
91052 Erlangen
Germany
Phone: +49 9131 84 0
[siemens.com/healthineers](https://www.siemens.com/healthineers)

Local Contact Information

Siemens Healthcare Diagnostics Inc.
Point of Care Diagnostics
2 Edgewater Drive
Norwood, MA 02062-4637
USA
Phone: +1 781-269-3000
[siemens.com/healthineers](https://www.siemens.com/healthineers)