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1. Important information - read prior to start-up

You have acquired a valuable RIESTER product manufactured in compliance with Directive 93/42/EEC for medical products and subject to continuous stringent quality control.

Should you have any queries, please contact the Company or your RIESTER Agent who will be pleased to assist you. For addresses see last page of these Operating Instructions. The address of your authorised RIESTER Agent will be supplied to you on request.

Please also note that correct and safe operation of products will only be guaranteed when RIESTER products and accessories are used throughout.

Significance of symbol on scales:
Caution, observe provided information

2. Purpose

The "Spirotest" is a newly developed unit allowing lung capacity to be measured without the use of water, affording a number of advantages over conventional, commercially available spirometers and thus greatly simplifying application. The unit is very handy and has a wide measuring range (1000 to 7000 cm³). It is designed for use in tests at sporting events and in the field of preventive medicine and can also be used for monitoring patients' own lung capacity.

3. Components (see figure)

- 3.1. Spirometer
- 3.2. Dial
- 3.3. Nozzle
- 3.4. Mouthpiece
- 3.5. Ring

4. Start-up and operation

- 4.1. Connect end of the mouthpiece (3.4.) to nozzle (3.3.).
- 4.2. Hold spirometer in one hand and check that the needle is set to "0".
- 4.3. If the needle is not set to "0" correct by rotating the upper outer ring (3.5.) in clockwise or counterclockwise direction.
- 4.4. Breathe in deeply and sit up straight. When your lungs are full, insert mouthpiece (3.4.) between your lips. Then breathe out continuously (within 4 - 5 seconds). This is very important to achieve precise results
- 4.5. When you have exhaled completely, read precise value (3.2).
- 4.6. Repeat this procedure 3 times. The maximum value of 3 measurements corresponds to your lung capacity.

Precautions!

- During the test, hold "Spirotest" in horizontal position. (If the unit is not held steady, the needle cannot rotate uniformly and measured results will not be correct).
- When using the unit, the small holes at the side of the upper section must not be covered with your hand.
- The "Spirotest" is a precision instrument similar to a clock, and must be handled with care.
- Only your doctor is in a position to analyse the values exactly.

5. Cleaning

Cleaning and/or disinfecting

The external surfaces of the spirometer can be cleaned with a moist cloth and can be disinfected with the following disinfecting agents: aldehyde, (formaldehyde, glutaraldehyde, aldehyde separators) or tensides.

All instrument components with the exception of glass components, may be disinfected with alcohol.

When using any of these substances, the instructions supplied by the manufacturer must be observed.

Caution!

If the unit is used regularly, water droplets and saliva will condense on the inside of the housing, impairing precision. After every 10 measurements remove the lower section and clean it with a cloth.

Disinfection

According to the ruling teaching (Tübingen Medical Products Test Centre), sterilisation is only specified for surgery.

6. Spare parts

Item No. 11230 Package of 50 pcs. plastic mouthpieces

Item No. 11232 Package of 1000 pcs. carton mouthpieces

7. Maintenance

These instruments and their accessories do not require any specific maintenance.

Should an instrument have to be examined for any specific reason whatsoever, please return it to the Company or an authorised RIESTER dealer in your area. Addresses to be supplied on request.

G U A R A N T E E

This device was manufactured in accordance with the most stringent quality requirements. Before leaving the factory, it underwent extensive final testing.

We are thus in the fortunate position to be able to offer you a guarantee extending for

2 years starting on the date of purchase

This guarantee covers any defects of the device which are demonstrably due to material or manufacturing faults. Your warranty will become null and void under the following circumstances: incorrect handling, non-compliance with Operating Instructions, use of other manufacturers' spares and accessories, unauthorised amendments or repairs and normal wear and tear.

It also excludes defects on the mouthpieces!

All defective parts of the device will be repaired or replaced free of charge within the guarantee period.

Claims made under the guarantee shall be valid only if the instrument is accompanied by this Guarantee Card, completely filled in and stamped by the dealer.

Please note that guarantee claims must be submitted within the guarantee period.

We will, of course, gladly perform testing or repair work after the guarantee period as well. Upon request, we will provide non-binding cost estimates for such work.

Should your instrument exhibit a defect during the guarantee period or require repair, please send the entire instrument, along with the filled-in Guarantee Card, to the following address:

Rudolf Riester GmbH
Abt. Reparaturen RR
Bruckstr. 31
D-72417 Jungingen
Germany

Date

Stamp and **signature** of the specialized dealer

Please detach this card on the perforated line!

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**Riester offers a large selection of products in the areas of**

Blood pressure measuring devices | Instruments for ENT, Ophthalmological instruments | Dermatological instruments | Thermometers | Stethoscopes | Head mirrors, Head lights, Examination lights | Laryngoscopes | Gynaecological instruments | Percussion hammers | Tuning forks | Products for blood stasis | Pulmonary pressure measuring devices | Dynamometers | Pressure infusion instruments | Veterinary instruments | Doctor's cases and bags

Detailed descriptions of the products can be found in the respective sections of the omnibus edition catalogue (Order No. 51232-50). Or online under www.riester.de.

**Rudolf Riester GmbH**

P.O. Box 35 • DE-72417 Jungingen

Germany

Tel.: +49 (0)74 77/92 70-0

Fax: +49 (0)74 77/92 70 70

info@riester.de • www.riester.de