Hinweis:

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ENGLISH

Foreword

Congratulations on purchasing this high quality Softlaser. Softlaser is an instrument which was mainly constructed to be easy and safe for ordinary people to use. The radiation that comes from what we call a Softlaser is controlled so that it neither generates heat nor cause human tissue to change.

The wave length of the laser beam is determined in such a way that the skin and the connective tissue are effectively and protectively penetrated. So that you can treat yourself with the Softlaser as successfully as possible, you must first have the cause of the problem clarified by a physician.

We'd like to wish you all the best with the BIOLAS, a speedy recovery and good health in the long term.

Laser beam

**Do not look into the beam when
using optical instruments.**

Laser class 2 M

Contents

Operating Components	2
Suggestions for use	4
Surface treatment	5
Laser acupuncture	6
What you need to know before using laser acupuncture	8
Changing and inserting the batteries	9
Cleaning the laser tip	9
Changing the laser tip	10
Guarantee	10
Technical data	10
Start up and service	10
Risks and tolerances	11
Contraindications	11
Declaration of conformity	12
Recommended reading	13
Safety information	14

Use in accordance with regulations

The BIOLAS Softlaser is used to treat the indications given in these instructions for use.

Please also refer to the instructions for use and safety information, for use in accordance with regulations.

The BIOLAS Softlaser and the operating components at a glance.


Battery-holder screw (10)

Battery-holder (9)


Housing (11)

With a coin or the blade of the multi-purpose, undo the battery-holder screw (10) until the battery holder (9) can be drawn out of the housing (11). Inserting the batteries/changing batteries: place the batteries in the holder (9) in accordance with the symbols (polarity). Insert the battery-holder into the housing (11) and carefully tighten the battery-holder screw.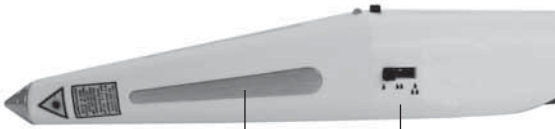

gold-plated contact surface (3)


Switch for Skin-type setting (4)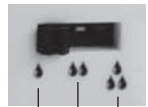

dry skin

moist skin

normal skin

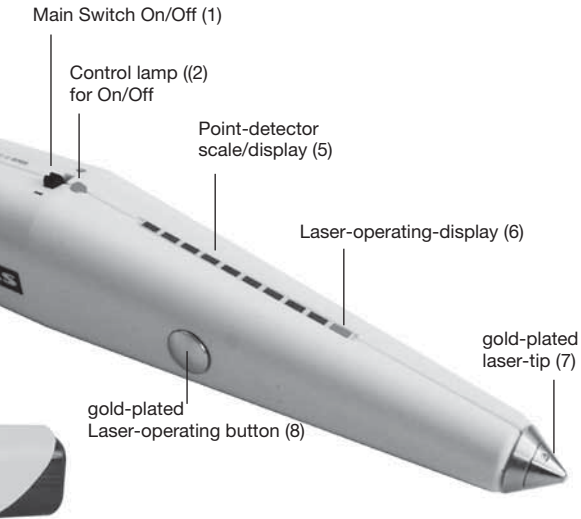To clean the laser nozzle, insert the multi-purpose tool supplied with the laser into one of the two axial apertures and undo the nozzle by rotating it anti-clockwise. Stick the tip of the tool through the laser nozzle opening and clean it with a brush or a soft cloth. Then screw the laser nozzle back on to the laser unit.

Suggestions for use

- Acne / spots
- Arthritis
- Eczema
- Fine lines
- Fever blisters
- Herpes simplex
- Small scars and wounds
- Cold sores
- Muscle tension
- Pustules
- Chapped skin
- Tooth ache
- Strains
- Gout
- 1 and 2 grade burns
- Tiredness
- Inflammation of joints
- Intestinal disorders
- Stomach pain

And much more.

BIOLAS corresponds to laser protection class 2M and has a visible laser beam and a variable laser beam cone.

You do not need to wear safety glasses or other protection. The eye's closing reflex provides it with sufficient protection.

Damage can only occur if you intentionally look into the beam for a long time. This should always be avoided.

Surface treatment

Operation

See illustrated chart “the operating elements at a glance”.

Switch the appliance on using the mains switch (1). The pilot light (2) will light up red. The BIOLAS is now ready for operation. Now hold the BIOLAS in your hand as you would hold a pencil (see diagram) and point the laser tip (7) at the area to be treated, at a distance of 1 to 2cm.

Do not touch the point that requires treatment with the laser tip if you are treating surface damage to the skin (e.g. herpes, wounds etc).

Press the laser operation button (8) once with your index finger (if you are right handed) or thumb (if you are left handed). Switch the laser on and the laser operation display (6) will light up red. After the prescribed radiation time, press the laser operation button again. This will switch the laser off. If you are using the laser for longer, the laser will switch off automatically after 10 minutes. If the prescribed treatment lasts for longer than 10 minutes (e.g. 12 minutes), just press the laser operation button again and switch off at the end of the remaining time (2 minutes). Switch off at the mains switch (1) when you have finished using the laser. The pilot light (2) will go out. The treatment can also be carried out by another person, because it is impossible to reach every body part yourself (e.g. on the back).

Please ensure that you switch off the appliance at the mains switch (1) after treatment, because electronic appliances use electricity constantly, even in standby mode, which could result in the batteries being empty within a few days.

Warning:

Never shine the laser beam in the eyes!

To find out how long you should use the laser, read the relevant specialist literature (see page 11) or ask your doctor, therapist or pharmacist.

Laser acupuncture

Operation

See illustrated chart: "The operating elements at a glance".

1. Switching on: Switch the appliance on using the mains switch (1). The inspection light (2) will light up red. The BIOLAS is now ready for use.

2. Find out your skin type: hold the BIOLAS in your hand like a pencil (see diagram). Touch the gold-plated contact plate (3) with your thumb (if you are right handed) or your index finger (if you are left handed). If you do not touch the contact plate there will be no display on the point locator scale (5). Now set the skin type adjustment switch (4) at the central position (2 drops = normal skin).

Determine your skin type by placing the laser tip (7) on the skin near any acupuncture point or an acupuncture point of your choosing (e.g. on your hand) with a light, even pressure and gliding it around. The green point locator scale (5) will now light up to various lengths (1 to 9 light points). If no fields or only a few fields light up, you have a different skin type.

If this is the case, set the skin type switch (4) at 1 drop = dry skin. If all of the fields of the green point locator scale (5) light up constantly, you have oily skin. Set the skin type determination switch (4) at 3 drops = oily skin. You normally only have to determine your skin type once, because generally your skin type will not change.

3. Point locator scale and laser acupuncture:

Now you can precisely determine the specific acupuncture point required for treatment. Proceed with the gold-plated laser tip (7) as described above, over the area of the skin where the acupuncture point is located. The gold-plated contact plates (3) must always be in contact with the thumb or index finger. The part of the skin where the point locator scale (5) shows the highest amplitude is the acupuncture point. Now hold the BIOLAS directly above this point and briefly press the laser operation button (8). The laser beam will switch on. Press the button again briefly after 15 to 60 seconds and the laser beam will switch off. During laser treatment, keep the BIOLAS as perpendicular to the surface of the skin as possible to get the best effect. The focal point of the laser

beam is exactly located at the laser tip outlet. By pressing the laser operation button (8), the green (5) will go out and the laser operation display (6) will light up red.

The laser beam is visible. Therefore, the skin around the acupuncture point will light up red for the duration of the acupuncture. The laser acupuncture can be carried out by another person, because it is impossible to reach every acupuncture point yourself. If this is the case, please ensure that the person carrying out the treatment always stays in contact with your skin with their free hand.

You can find examples of use with the most important acupuncture areas on page 32.

At the end of the laser acupuncture, please switch off at the mains switch (1). The pilot light (2) will go out. Please ensure that you switch off the appliance at the mains switch (1) after treatment, because electronic appliances use electricity constantly, even in standby mode, which could result in the batteries being empty within a few days.

Warning! Never shine the laser beam in the eyes!

What you need to know before using laser acupuncture.

Duration of treatment

Because everyone is different, laser acupuncture will have a different effect depending on the individual. It is also impossible to give precise information on the treatment period. Our experience will give you an idea:

depending on the level of pain, you can laser the appropriate acupuncture point two to three times a day, for between 15 and 60 seconds. Be patient during the treatment and allow a period of three to four weeks for physical symptoms to improve. Results won't appear overnight, especially if you are treating a nervous disorder. You can usually rely on your body's reactions and you will know yourself what length of treatment is beneficial to you. The relevant specialist literature will provide more information on the location of the acupuncture points and their range of effect (see literature recommendations page 14) or ask your doctor, therapist or pharmacist.

Create a relaxed atmosphere!

- Make sure that you won't be disturbed.
- Relax in a sitting or lying position and prepare yourself mentally for the treatment and its effects.
- Quiet meditation or relaxation music is very good for creating a harmonious atmosphere.
- Don't use the laser when you are short of time and make sure you concentrate on the individual acupuncture points.
- You can carry out the treatment two to three times a day if you are suffering acute pain.
- Because the meridians, apart from the conception and governing vessels, are arranged in pairs, you should laser the acupuncture points on both sides of the body, one after the other.

Please note:

Self-treatment with BIOLAS does not replace essential medical treatments and interventions under any circumstances. However, the treatment can be successful in conjunction with other treatments in many cases.

Care and instructions

Changing and inserting the batteries

See illustrated chart "the operating elements at a glance".

The BIOLAS is designed for use with AAA LR03 batteries (1.5 volt). Please only use leak-proof batteries. This will keep your BIOLAS working, problem-free for many years. It is not suitable for use with rechargeable batteries or with a power pack.

Check the condition of the batteries

After the laser operation button has been pressed, the timer will switch off after 10 minutes = the batteries are full. The timer will switch off immediately or after 1 to 2 minutes = the batteries are empty.

Changing the batteries

Using a multi-purpose pen or a small coin, unscrew the battery compartment screw (10), until the battery compartment (9) can be removed from the casing (11).

Remove the used batteries and insert four new batteries (Micro LR03/AAA) into the battery compartment, according to the diagram and observing correct polarity. Push the battery compartment (9) back in the housing (11) and carefully and firmly screw on the battery compartment screw (10).

Change used batteries immediately and dispose of them in an environmentally friendly way.

Cleaning the laser tip

See illustrated chart "the operating elements at a glance".

If the laser tip is pressed on the skin, it will become covered with skin particles or oil. This prevents the laser beam from working at full strength. Make sure that you clean the laser tip at regular intervals.

To clean the laser outlet, insert the multi-purpose pen provided into one of the cross-holes and undo the tip in an anti-clockwise direction.

Clean the laser outlet with a soft cloth or a cotton wool bud. Then screw the laser tip back onto the appliance.

Warning! In order to avoid damage only put on the laser tip by hand.

Changing the laser tip

If necessary, you can add a laser tip with an optical wave guide (accessories).

To remove the laser tip, insert the multi-purpose pen provided into one of the two cross-holes and undo the tip in an anti-clockwise direction. Then screw the laser tip back onto the appliance.

Warning! In order to avoid damage only put on the laser tip by hand.

Guarantee

The BIOLAS is guaranteed for two years, starting from the date of sale (please retain your receipt).

Within the guarantee period, we will repair defects free of charge, if these defects are proven to be caused by manufacturing or material errors. Parts that are subject to wear and tear are excluded from the guarantee. In addition, the guarantee claim is not valid for incorrect operation or failure to adhere to the instructions for use, the use of astringent cleaning products, unauthorised interventions or changes to the appliance.

Technical data

Operating voltage: 6V direct current via 4 micro batteries type Micro LR03/AAA (1.5V).

Laser diode: visible red-light laser diode.

Wavelength: Approx. 650nm.

Beam divergence angle: Approx, 17 degrees.

Laser output: < 5 mW. Divergent continuous beam

Laser protection class: 2M

Medical appliance class: II a

Start up and service

The BIOLAS is ready for use as soon as the batteries have been inserted. The casing is welded, so laser output can only be adjusted by forcing the casing open. If this happens, the guarantee shall cease to be valid and the manufacturer shall accept no responsibility for any damage caused by excessively powerful laser beams. Should any technical problems arise, the appliance should be sent back to the manufacturer or handed into the appropriate specialist retailer immediately.

Risks and tolerances

Laser light is damaging to the eyes. Never look into the beam, even with optical instruments, because optical instruments can bring the divergent rays back together again. Additional protection is not required.

The casing is made from ABS LP 227 S and fulfils the requirements of the foodstuffs and consumer goods law of 10/04/1992, subsequent changes to this law and the BgVV (formerly BGA) recommendation VI, 1/12/1996 version.

Contraindications

The metal components of BIOLAS are hard gold-plated and provide the best possible protection against micro-organisms. For galvanisation, the gold layer has to have a nickel base coat. It is therefore possible that people with a nickel allergy could suffer an allergic reaction.

- **Patients with pacemakers:** No laser beam around the ribcage or pacemaker
- **Pregnancy:** No laser beam in the stomach area / lower thorax / certain acupuncture points (!)
- **Patients with tumours:** No laser beam on the tumour or in the case of suspicion of relapse or metastases
- **EC – no laser treatment of people with epiphysis** (growth areas of bones) and hollow bones in children
- **No laser treatment of the thyroids, ovaries and testicles**
- **No laser treatment for epilepsy** (oligoepilepsy)

EC declaration of conformity

In accordance with EC guideline 93/42/EWG appendix VII with V

It was executed a conformity evaluation process, in accordance with appendix V of EC guideline 93/42/EWG and has been proven by LGA InterCert (identification number 1275).

This declaration is valid for all of the abovementioned products bearing the CE identification and that originate from the following factories:

The declaration of Conformity is available on request

GML German Medical Laser GmbH
Reutlinger Str. 14
75179 Pforzheim

Tel: + 49 (0) 7231 - 13 95 69-0
Fax: + 49 (0) 7231 - 13 95 69-50

Email: info@gm-laser.de

Recommended reading

You are bound to have benefited from using your BIOLAS low level laser and want to know more about laser therapy and laser acupuncture.

Maybe you are also interested in the scientific theory of the Chinese needle treatment, Chinese teachings on energy and other types of healing.

Here are some examples of relevant books you can find in bookshops:

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GML:

Guidebook for Softlaser-Therapy.

You can use these books to learn all about laser therapy: from how to work with lasers and the properties of laser light to the results of investigations, applications, symptoms and diseases.

Wertsch/Schrecke/Küstner:

Akupunkturatlas

[Acupuncture atlas]

WBV, ISBN 3-92-1988-39-X

Gerhard Leibold:

Akupressur

[Akupressure]

Falken-Taschenbuchverlag, ISBN 3-635-6005-9

These publications give you lots of examples of healing, detailed drawings with the location of the acupuncture points that you can obviously be used for treatment with BIOLAS - and they provide clear explanations on energy teaching.

Safety advice for working with BIOLAS

1. Eye protection:

Even at low energy, laser beams can damage the retina. Never shine the laser beam into the eyes and do not look into the beam, even with optical instruments such as glasses or lenses etc...

2. Protecting children:

Always keep BIOLAS out of the reach of children. Children cannot estimate the dangers of a laser appliance.

3. Protection against infections:

Wounds and bacterial infections may only be treated with surface laser beams at a distance of 1 to 2cm. If the laser tip comes into contact with an infected site, unscrew the laser tip and use a normal disinfectant to clean it. Before using laser treatment on an open wound, the laser tip should always be disinfected so that germs are not transferred to the wound if they accidentally come into contact.

4. Protecting the environment:

Return used batteries to a specialist retailer. You'll be contributing to recycling and protecting the environment.

The appliance should be disposed of appropriately when it reaches the end of its life. Ask your specialist trader or manufacturer.

GML German Medical Laser GmbH

Reutlinger Str. 14
75179 Pforzheim

Tel: + 49 (0) 7231 - 13 95 69-0
Fax: + 49 (0) 7231 - 13 95 69-50

Email: info@gm-laser.de
www.gm-laser.de

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Hippokrates Verlag GmbH, Stuttgart ISBN 3-7773-1019-0