



SurGuard2™

Safety Hypodermic Needle



Avoid a Stick with a Simple Click

- *Comfortable one-hand activation*
- *Secure injection*
- *Secure safety activation*
- *Easy handling*



Comfortable one-hand activation

Simple safety mechanism allows sheath activation with one hand.

Secure injection

Fixed bevel orientation (bevel up when shield is 90° to the right) assures secure injection.

Secure safety activation

Audible click and clear sheath window ensure confirmation of secure locking of the needle in the shield.

Easy handling

Stable and flexible sheath with hinge and finger grip facilitates easy handling of safety activation.

Additional Features and Benefits:

- Easy puncture with ultra sharp lancet point
- Smooth needle movement with high quality silicon coating
- Easy injection with Terumo thin walled needles
- Wide range of products

Products and specifications

CE
0197


Product Code	Colour	Cannula ø	Cannula Length
SG2+1825	Pink	18G 1.20 mm	1" 25 mm
SG2+1838	Pink	18G 1.20 mm	1 ½" 38 mm
SG2+1925	Beige	19G 1.10 mm	1" 25 mm
SG2+1938	Beige	19G 1.10 mm	1 ½" 38 mm
SG2+2025	Yellow	20G 0.90 mm	1" 25 mm
SG2+2038	Yellow	20G 0.90 mm	1 ½" 38 mm
SG2+2125	Green	21G 0.80 mm	1" 25 mm
SG2+2138	Green	21G 0.80 mm	1 ½" 38 mm
SG2+2225	Black	22G 0.70 mm	1" 25 mm
SG2+2238	Black	22G 0.70 mm	1 ½" 38 mm
SG2+2325	Blue	23G 0.65 mm	1" 25 mm
SG2+2338	Blue	23G 0.65 mm	1 ½" 38 mm
SG2+2516	Orange	25G 0.50 mm	5/8" 16 mm
SG2+2525	Orange	25G 0.50 mm	1" 25 mm
SG2+2538	Orange	25G 0.50 mm	1 ½" 38 mm
SG2+2609	Dark brown	26G 0.45 mm	3/8" 9 mm
SG2+2613	Dark brown	26G 0.45 mm	1/2" 13 mm
SG2+2713	Grey	27G 0.40 mm	1/2" 13 mm
SG2+3013	Yellow	30G 0.30 mm	1/2" 13 mm

Shipping Carton:
800 pieces (8 x unit box)


Sterilisation
with e-beam




Instruction for use – after injection

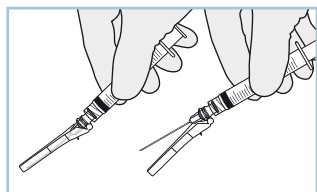


Position shield about 40°-45°. With a FIRM, quick motion, press down against a flat surface until an audible click is heard.

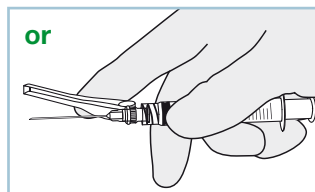

With a FIRM, quick motion press with your finger the shield into place until an audible click is heard.

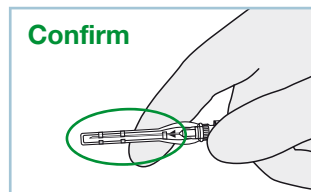

Confirm that the shield is fully engaged.

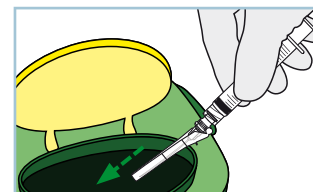

Dispose the SurGuard2™ after use into a disposal box.



Terumo Corporation
44-1, 2-chome, Hatagaya
Shibuya-ku, Tokyo 151-0072,
Japan
Tel.: +81 3 3374 8111
Fax: +81 3 3374 8196

Laboratoires Terumo France S.A.
Parc Ariane - Bât. Uranus
1, rue Hélène Boucher
78284 Guyancourt Cedex, France
Tel.: +33 130 96 13 00
Fax: +33 130 43 60 85

Terumo Italia S.r.l.
Via Simone Martini, 143/145
00142 Roma RM,
Italy
Tel.: +39 0651 96 14 20
Fax: +39 0650 30 407

**Terumo Europe N.V.
Moscow Representative Office**
BC Northern Tower
13Fr., 10 Testovskaya Street
Moscow 123317, Russia
Tel.: + 7 495 988 4740
Fax: + 7 495 988 4739

European Headquarters

Terumo Europe N.V.
Interleuvenlaan 40
3001 Leuven, Belgium
Tel.: +32 16 38 12 11
Fax: +32 16 40 02 49

European Offices

**Terumo Europe N.V.
Benelux Sales Division**
Interleuvenlaan 40
3001 Leuven, Belgium
Tel.: +32 16 39 25 80
Fax: +32 16 39 25 99
The Netherlands:
Tel.: 0800 0220396
Fax: 0800 0220414

Terumo Deutschland GmbH

Ludwig-Erhard-Strasse 6
65760 Eschborn, Germany
Tel.: +49 6196 80 230
Fax: +49 6196 80 23 200

Terumo Deutschland GmbH

Zweigniederlassung Spreitenbach
Bodenackerstrasse 3
8957 Spreitenbach, Switzerland
Tel.: +41 56 419 10 10
Fax: +41 56 419 10 11

Terumo Europe España SL

Avda. Juan Carlos I, Nº 13 - 7ª Planta
Edificio Torre La Garena
28806 Alcalá de Henares (Madrid),
Spain
Tel.: +34 9021 01 298
Fax: +34 9021 01 358

Terumo Sweden AB

Nya Varvet, Byggnad 90
Sven Källfelts Gata 18
426 71 Västra Frölunda,
Sweden
Tel.: +46 3174 85 880
Fax: +46 3174 85 890

Terumo Denmark

Filial of Terumo Sweden AB
Tel.: +45 7020 93 80
Fax: +45 7020 94 80

Terumo UK Ltd

Tamesis
The Causeway
Egham, Surrey
TW20 9AW, United Kingdom
Tel.: +44 1784 476 200
Fax: +44 1784 476 201

**Addresses subject to change,
please consult our Website.**

© Registered Trademark
Published by Terumo Europe N.V.
HO 36GB-0907FK-II(10.11)E